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產品型號	Volume	產品	Type
MDCSF01S2516	1mL	Safety Clip type Syringe with needle 25G x 16mm	Luer Slip type
MDCSF01S2613	1mL	Safety Clip type Syringe with needle 26G x 13mm	Luer Slip type
MDCSF01S2713	1mL	Safety Clip type Syringe with needle 27G x 13mm	Luer Slip type
MDCSF03L2325	3mL	Safety Clip type Syringe with needle 23G x 25mm	Luer Lock type

MDCSF03L2332	3mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Lock type
MDCSF03L2425	3mL	Safety Clip type Syringe with needle 24G x 25mm	Luer Lock type
MDCSF03S2325	3mL	Safety Clip type Syringe with needle 23G x 25mm	Luer Slip type
MDCSF03S2332	3mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Slip type
MDCSF03S2425	3mL	Safety Clip type Syringe with needle 24G x 25mm	Luer Slip type
MDCSF05L2138	5mL	Safety Clip type Syringe with needle 21G x 38mm	Luer Lock type
MDCSF05L2232	5mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Lock type
MDCSF05L2332	5mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Lock type
MDCSF05S2325	5mL	Safety Clip type Syringe with needle 23G x 25mm	Luer Slip type
MDCSF05S2332	5mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Slip type
MDCSF10L2138	10mL	Safety Clip type Syringe with needle 21G x 38mm	Luer Lock type
MDCSF10L2238	10mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Lock type
MDCSF10L2125	10mL	Safety Clip type Syringe with needle 21G x 25mm	Luer Lock type
MDCSF10L2232	10mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Lock type
MDCSF10S2138	10mL	Safety Clip type Syringe with needle 21G x 38mm	Luer Slip type
MDCSF10S2232	10mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Slip type
MDCSF10S2238	10mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Slip type
MDCSF10S2325	10mL	Safety Clip type Syringe with needle 23G x 25mm	Luer Slip type
MDCSF20S2232	20mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Slip type
MDCSF20S2238	20mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Slip type
MDCSF20S2332	20mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Slip type
MDCSF20S2338	20mL	Safety Clip type Syringe with needle 23G x 38mm	Luer Slip type
MDCSF20L2232	20mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Lock type
MDCSF20L2238	20mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Lock type
MDCSF20L2332	20mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Lock type
MDCSF20L2338	20mL	Safety Clip type Syringe with needle 23G x 38mm	Luer Lock type
MDCSF30S2232	30mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Slip type
MDCSF30S2238	30mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Slip type
MDCSF30S2332	30mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Slip type
MDCSF30S2338	30mL	Safety Clip type Syringe with needle 23G x 38mm	Luer Slip type
MDCSF30L2232	30mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Lock type
MDCSF30L2238	30mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Lock type
MDCSF30L2332	30mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Lock type
MDCSF30L2338	30mL	Safety Clip type Syringe with needle 23G x 38mm	Luer Lock type
MDCSF50S2232	50mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Slip type
MDCSF50S2238	50mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Slip type
MDCSF50S2332	50mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Slip type
MDCSF50S2338	50mL	Safety Clip type Syringe with needle 23G x 38mm	Luer Slip type
MDCSF50L2232	50mL	Safety Clip type Syringe with needle 22G x 32mm	Luer Lock type

MDCSF50L2238	50mL	Safety Clip type Syringe with needle 22G x 38mm	Luer Lock type
MDCSF50L2332	50mL	Safety Clip type Syringe with needle 23G x 32mm	Luer Lock type
MDCSF50L2338	50mL	Safety Clip type Syringe with needle 23G x 38mm	Luer Lock type

製造業者名稱：Yangzhou Medline Industry Co., Ltd.

製造業者地址：No. 1, Huafa Road, Economic Development Zone, Yangzhou, Jiangsu, 225000, China

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