

"泰爾茂" 一次性使用注射器(附針/不附針)

"Terumo" Disposable Syringe (with/without needle)

衛部醫器陸輸字第 000774 號

使用前請務必詳閱原廠之使用說明書並遵照指示使用。

< 產品用途 >

該產品適用於抽吸液體或在注入液體後與注射針配套使用立即對人體注射。

< 警告 / 注意事項 >

警告


- 產品使用前先旋緊注射針後再進行注射等其他對應操作
- 開啟單一包裝後請立即使用，使用後請適當安全地棄置，以減少感染危機。
- 謹慎處理以避免針刺，使用後請勿把針頭護套復位。

注意事項

- 以環氧乙烷滅菌，請勿再滅菌或重複使用。
- 如果單一包裝或產品已損壞或污染則請勿使用。
- 請勿用於顯影劑的高壓注射。
- 請勿存放於極端溫度和濕度的環境，避免直射陽光。
- 請依照產品用途使用，請勿將產品用於藥品存儲之用途。

< 使用方法 >

打開膠套包裝 (圖 1)：向回撕開最上層的包裝。

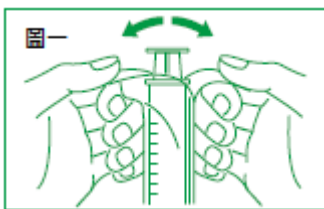

< 產品規格 >

注射器規格：

Volume: 1ml, 3ml, 5ml, 10ml, 20ml, 50ml, 60ml

Tip: Luer Lock (1ml, 3ml, 5ml, 10ml, 20ml, 50ml, 60ml), Luer Slip (1ml, 3ml, 5ml, 10ml, 20ml, 50ml, 60ml), Eccentric Luer Slip (20ml, 50ml, 60ml), Eccentric Luer Lock (20ml, 50ml, 60ml)

注射針規格

Gauge Size	Color code	Needle Length
18G	Pink	5/16"-1 1/2" (8mm-40mm)
19G	Creamy yellow	
20G	Yellow	
21G	Green	
22G	Black	
23G	Blue	
24G	Purple	
25G	Orange	
26G	Brown	
27G	Grey	
28G	Blue-green	
29G	Red	
30G	Light yellow	

*因應客戶需求增加產品型號：

產品型號與其規格

產品型號	Volume	產品	Type
MDSS01S	1mL	Syringe without needle	Luer Slip type
MDSS01S2516	1mL	Syringe with needle 25G x 16mm	Luer Slip type
MDSS01S2525	1mL	Syringe with needle 25G x 25mm	Luer Slip type
MDSS01S2613	1mL	Syringe with needle 26G x 13mm	Luer Slip type
MDSS01S2713	1mL	Syringe with needle 27G x 13mm	Luer Slip type
MDSS01L	1mL	Syringe without needle	Luer Lock type
MDSS01L2516	1mL	Syringe with needle 25G x 16mm	Luer Lock type
MDSS01L2525	1mL	Syringe with needle 25G x 25mm	Luer Lock type
MDSS01L2613	1mL	Syringe with needle 26G x 13mm	Luer Lock type
MDSS01L2713	1mL	Syringe with needle 27G x 13mm	Luer Lock type
MDSS03L	3mL	Syringe without needle	Luer Lock type
MDSS03L2425	3mL	Syringe with needle 24G x 25mm	Luer Lock type
MDSS03L2332	3mL	Syringe with needle 23G x 32mm	Luer Lock type
MDSS03S	3mL	Syringe without needle	Luer Slip type
MDSS03S2325	3mL	Syringe with needle 23G x 25mm	Luer Slip type
MDSS03S2332	3mL	Syringe with needle 23G x 32mm	Luer Slip type
MDSS03S2425	3mL	Syringe with needle 24G x 25mm	Luer Slip type
MDSS05L	5mL	Syringe without needle	Luer Lock type

MDSS05L2138	5mL	Syringe with needle 21G x 38mm	Luer Lock type
MDSS05L2232	5mL	Syringe with needle 22G x 32mm	Luer Lock type
MDSS05L2332	5mL	Syringe with needle 23G x 32mm	Luer Lock type
MDSS05S	5mL	Syringe without needle	Luer Slip type
MDSS05S2325	5mL	Syringe with needle 23G x 25mm	Luer Slip type
MDSS05S2332	5mL	Syringe with needle 23G x 32mm	Luer Slip type
MDSS10ES	10mL	Syringe without needle	Eccentric Luer Slip type
MDSS10L	10mL	Syringe without needle	Luer Lock type
MDSS10L2138	10mL	Syringe with needle 21G x 38mm	Luer Lock type
MDSS10L2238	10mL	Syringe with needle 22G x 38mm	Luer Lock type
MDSS10S	10mL	Syringe without needle	Luer Slip type
MDSS10S2138	10mL	Syringe with needle 21G x 38mm	Luer Slip type
MDSS10S2232	10mL	Syringe with needle 22G x 32mm	Luer Slip type
MDSS10S2238	10mL	Syringe with needle 22G x 38mm	Luer Slip type
MDSS10S2325	10mL	Syringe with needle 23G x 25mm	Luer Slip type
MDSS20S	20mL	Syringe without needle	Luer Slip type
MDSS20ES	20mL	Syringe without needle	Eccentric Luer Slip type
MDSS20L	20mL	Syringe without needle	Luer Lock type
MDSS20EL	20mL	Syringe without needle	Eccentric Luer Lock type
MDSS50S	50mL	Syringe without needle	Luer Slip type
MDSS50ES	50mL	Syringe without needle	Eccentric Luer Slip type
MDSS50L	50mL	Syringe without needle	Luer Lock type
MDSS50EL	50mL	Syringe without needle	Eccentric Luer Lock type
MDSS60S	60mL	Syringe without needle	Luer Slip type
MDSS60ES	60mL	Syringe without needle	Eccentric Luer Slip type
MDSS60L	60mL	Syringe without needle	Luer Lock type
MDSS60EL	60mL	Syringe without needle	Eccentric Luer Lock type

製造業者名稱：Yangzhou Medline Industry Co., Ltd.

製造業者地址：No.108, Jinshan Road Economic Development Zone, 225009, Yangzhou, Jiangsu,
China

醫療器材商名稱：台灣泰爾茂醫療產品股份有限公司

醫療器材商地址：臺北市中正區忠孝西路一段4號8樓